




Key Points

- Use the adjust line minus button to reduce the length or width of the measurement line to ZERO. This allows you to document a wound that is “CLOSED” with zero measurements.
 - Note: Users can also change measurements to ZERO on the web application
- Navigate to “Mark wound as healed” when the wound is considered HEALED (currently this can only be done on mobile, not on the web app)
 - Note: A “closed” wound does NOT necessarily mean a “healed” wound . . . Often it will take weeks or months for a wound to heal after closing.

Process to document a wound that is “CLOSED” with zero measurements


Draw short line for LENGTH

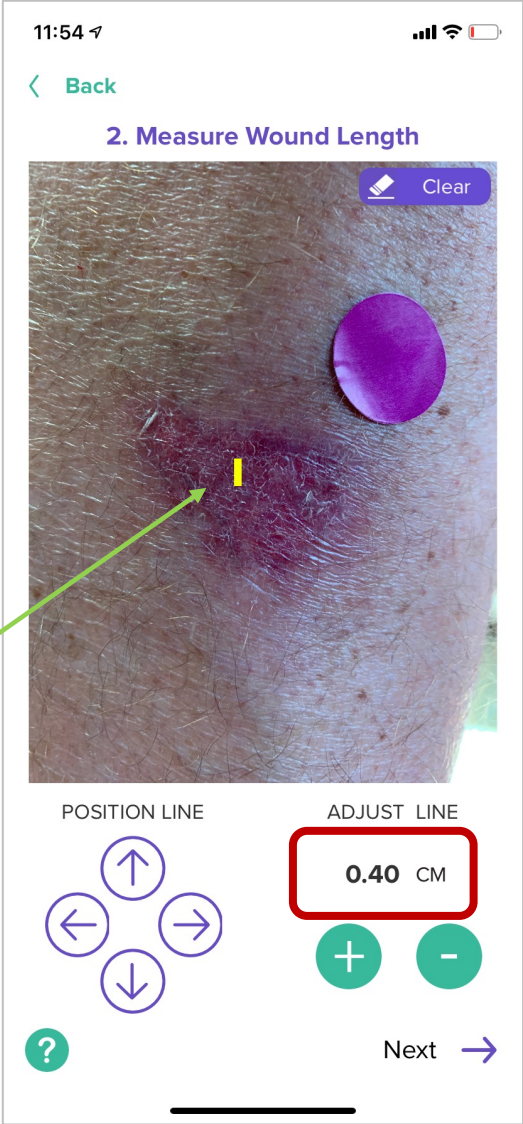

Adjust line to 0.00 cm LENGTH using minus button



Draw short line for WIDTH

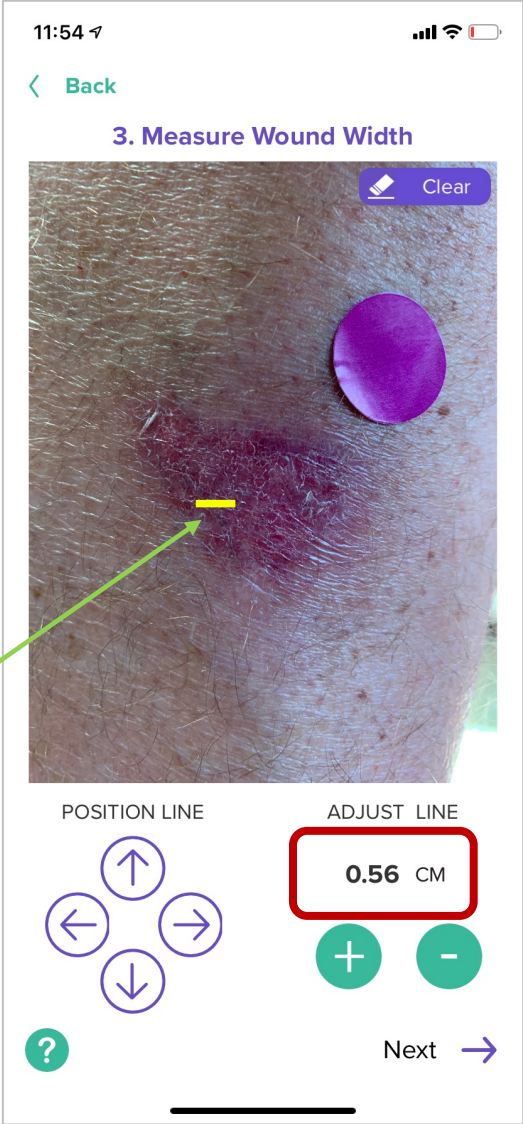

Adjust line to 0.00 cm WIDTH using minus button



Process to document a wound that is “CLOSED” with zero measurements (continued)

Mark at center of wound

11:55

Back

4. Mark Deepest Part of Wound



Clear

Tap pink dot to mark the location of the deepest part of wound

?

Next

Enter zero for DEPTH

11:55

Back

Wound Depth (centimeters)

Enter a measurement:

0

CONTINUE

11:55

Back

5. Confirm and Save



Length (Head to Toe) cm : 0.00

Width (Hip to Hip) cm : 0.00

Depth cm : 0

• = Deepest part of wound




Save

Process mark a wound as healed

Select – Mark wound as healed

Wounds that show are wound area = 0 cm²
Tap the SELECT button for wound

Tap MARK HEALED button
then click Back Button
IMPORTANT – IGNORE
processing symbol

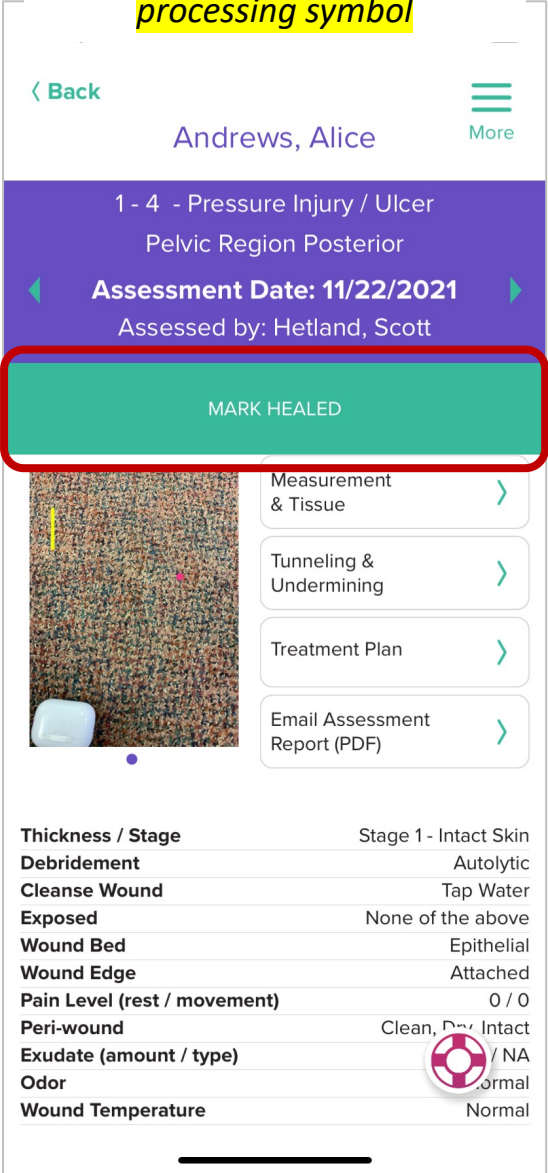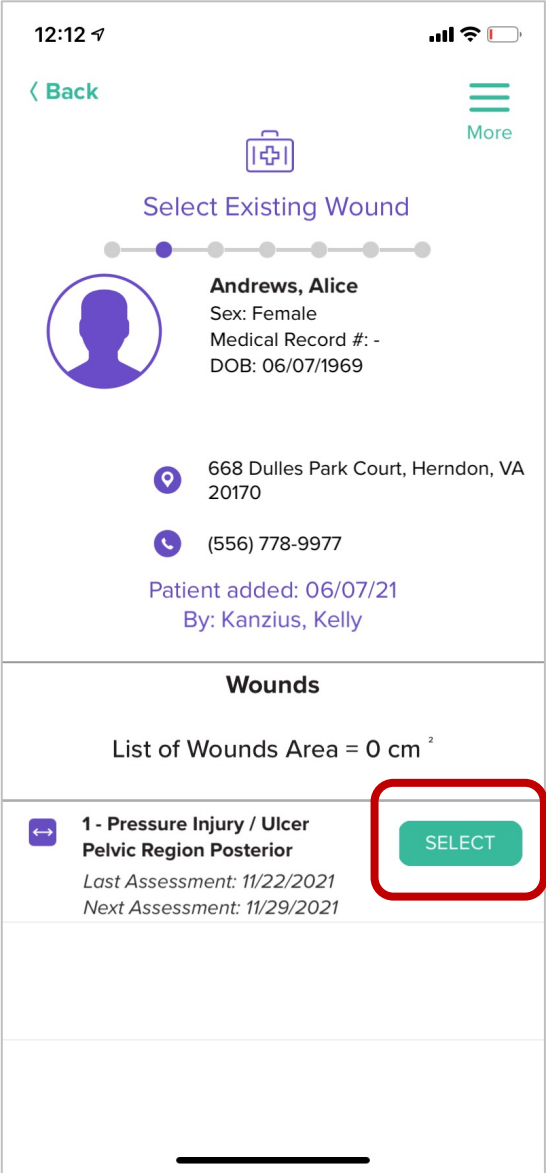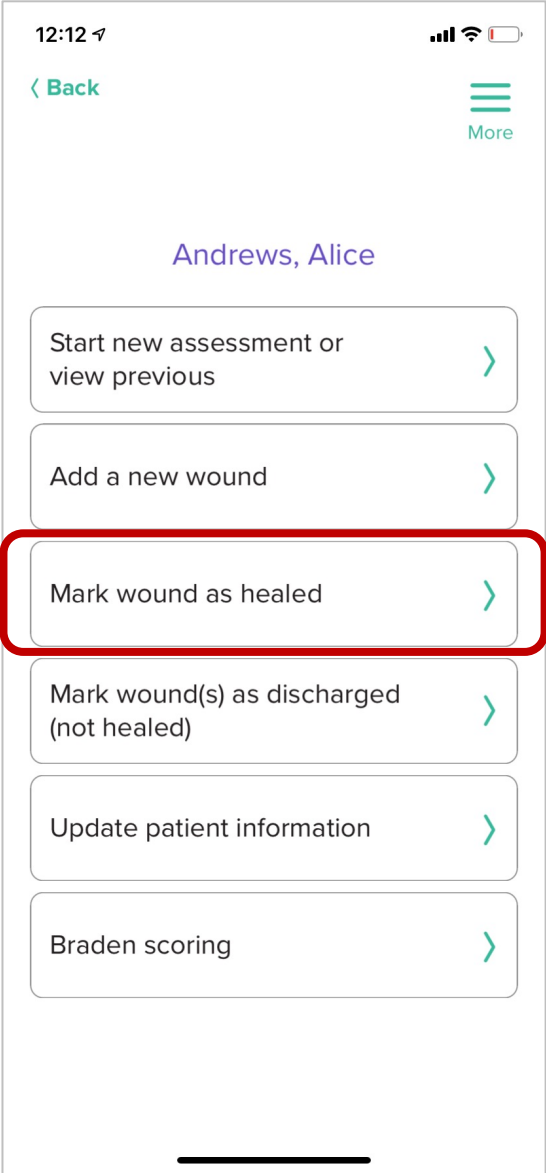

Notice "H"
indicates
wound is
marked as
Healed