





Key Points

- Be consistent Take all photos with the top of your device pointed in the direction of the head of the patient
- Do not take photos where the top of the phone is pointed at either hip as this will "flip" the length and width measurements
 - Note: Photos taken this way cannot be later rotated
- How you have set your portrait orientation lock (on or off) and camera ratio (1:1, 4:3 or 16:9) does not affect photos taken within Archangel.

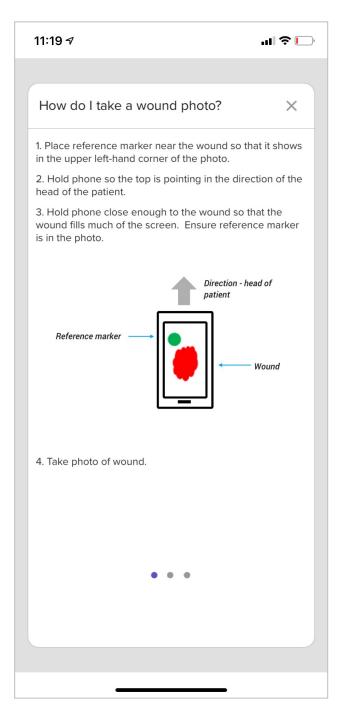

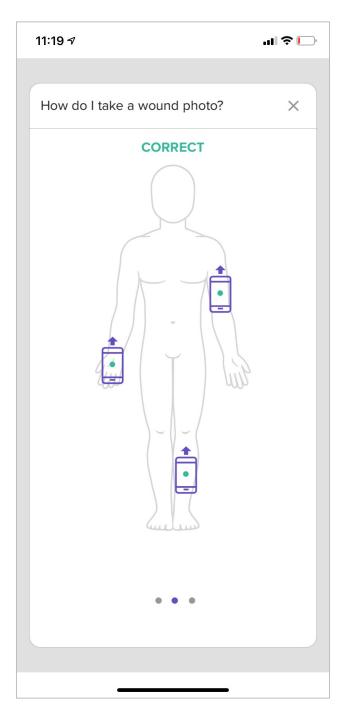


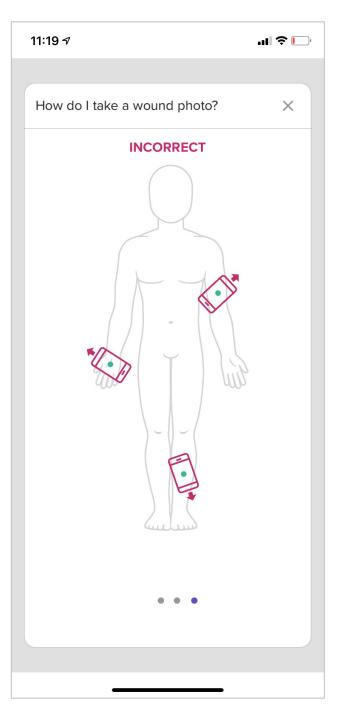

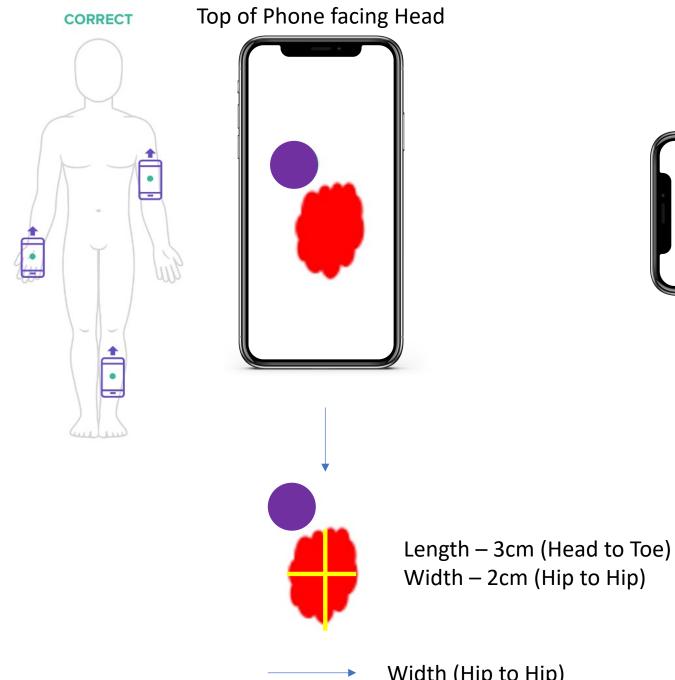


Top of phone facing left hip

Incorrect

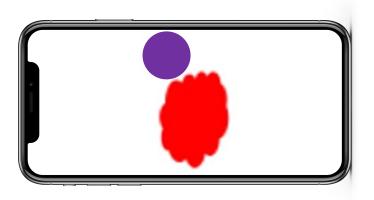

PROBLEM: Wounds taken this way shift the photo 90 degrees and the resulting measurements are "flipped".

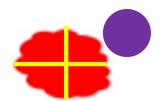


Length – 2cm Width – 3cm

Width (Hip to Hip)



Taking photo while in bed or in a chair

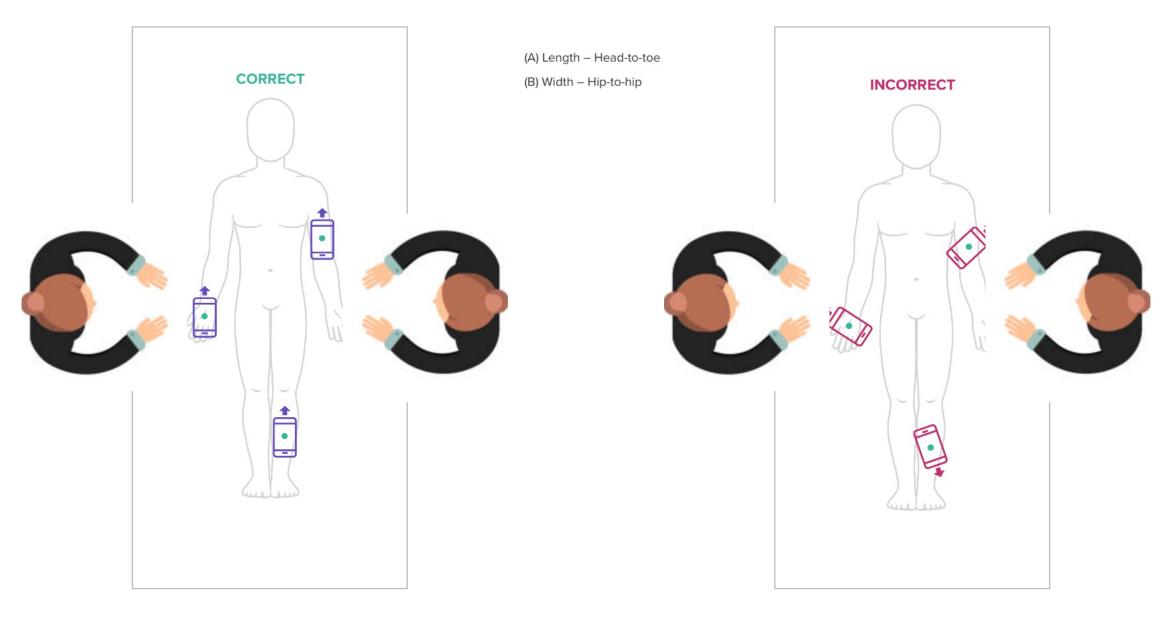

Same for patient on stomach or on side – position phone pointing to head

In the case of wound on bottom of foot Hold phone so the top of phone is in same direction of top of foot






How do I measure a wound?



LENGTH AND WIDTH


Measure the length "head-to-toe" at the longest point (A).

Measure the width side-to-side at the widest point (B) that is perpendicular to the length forming a '+'.

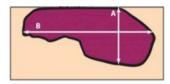

DEPTH

Measure the depth at the deepest point of the wound.

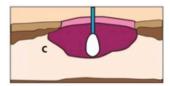

SUMMARY

- (A) Length Head-to-toe
- (B) Width Hip-to-hip
- (C) Depth Deepest Point